SILLA DE BAÑO

moem

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EC DECLARATION OF CONFORMITY

Declara que el producto / *This is to certify that the product*

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*Complies with the requirements of RD 1591/2009 of 16th October, that regulates medical devices and implementing Directive 93/42/ECC, modified by Directive 2007/47/EC.
Declaration in accordance with Annex VII of RD 1591/2009. Medical devices class I*

Reference standard: **UNE-EN 12183 “Manual wheelchairs. Requirement and test methods”.**

Viana, 20 de marzo de 2017

Firmado / *Signed:*

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Director Técnico / *Technical Manager*

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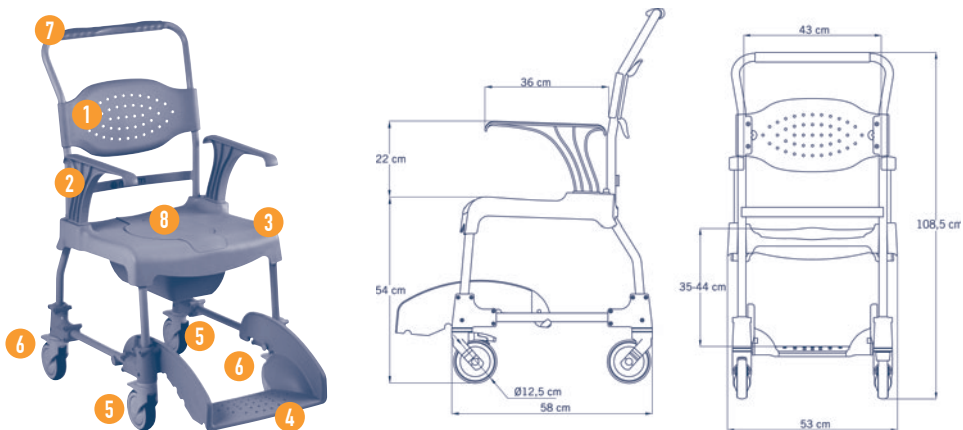
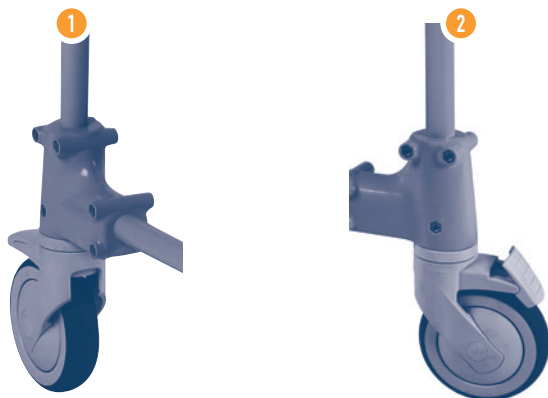
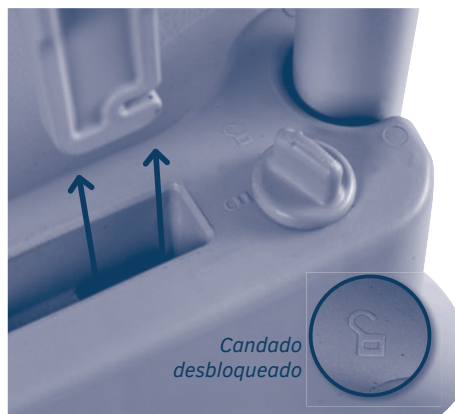
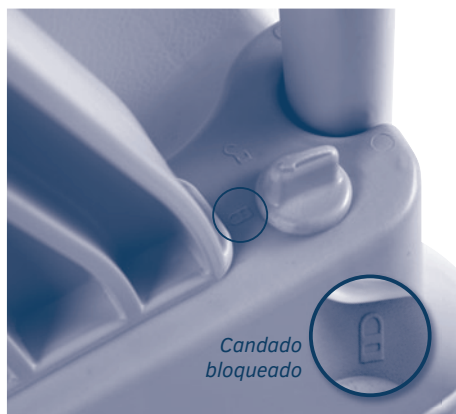
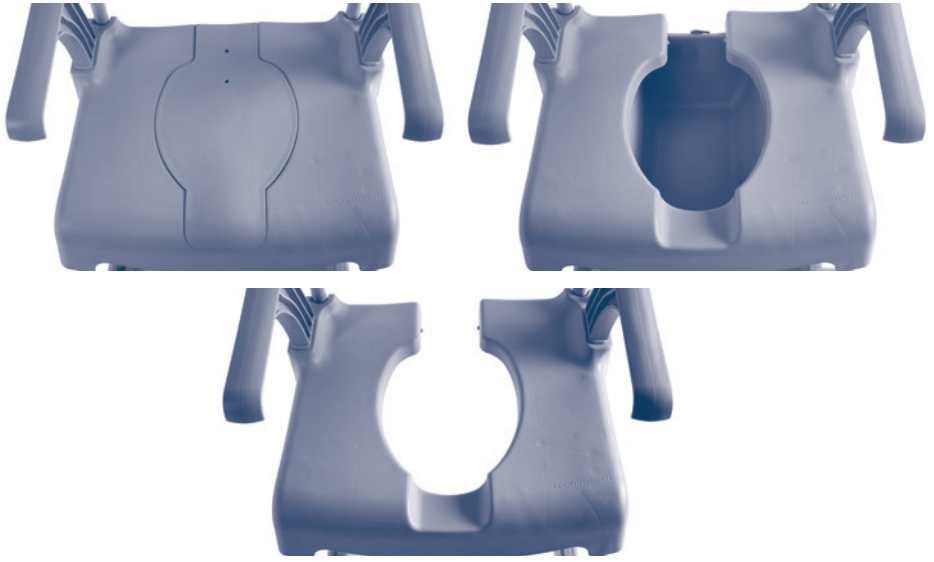
1/ GENERAL

 **To avoid damage during handling or use of the chair please read this manual carefully.**

MOEM is a shower and toilet chair designed to make hygiene easier for people with mobility difficulties. It is intended for use in a shower, at the washbasin or over a toilet, and can also be used to transport patients.

MOEM can be used with patients weighing **up to 150 kg**.

MOEM is considered a class I healthcare product, under current Spanish healthcare product regulations.

 **All the images referred to are on pages 4 and 5 of this manual.**

2/ DESCRIPTION AND MEASUREMENTS See image **A** (page 4)

- | | |
|---|---|
|  1 Back |  5 Wheels with brake |
|  2 Armrest |  6 Wheels without brake |
|  3 Seat |  7 Carer handhold |
|  4 Footrest |  8 Seat cover |

3/ HANDLING See image **B** (page 4)**Getting over small steps**

If the chair is empty, put your foot on the wheel fixing component.

If the chair is occupied, use the care assistant's handhold with the chair facing backwards.

Brake the chair (image **B** **1** - **B** **2**)

Lock the wheels by stepping on the brake pedal. The chair is more stable if the wheels are turned outwards.

Transfer

Make sure the brake is on the wheels before starting to transfer the patient to another support (toilet, bed...). Remove the armrest beforehand if necessary.

4/ ARMREST See image **C** (page 4)

Armrest can be extracted using the wheel.

5/ SEAT COVER See image **D** (page 4)

Easy to extract.

6/ FOOTREST See image **E** (page 5)

Detachable and easy to use.
Adjustable to 5 positions.

7/ GUARANTEE

2 year guarantee against manufacturing defects.
The guarantee and liability expire in the event of interventions by unauthorised staff or failure to follow the user and maintenance instructions.

8/ MAINTENANCE

Clean regularly with non-abrasive products.
Rinse and dry.
Keep the wheels and brakes free of dirt and soap.

9/ SAFETY

 **Do not use the chair if you notice any defects.**

Failure to do this may result in material damage or physical injury.
Do not stand on the footrest (chair may overturn).
Take care with the seat cover and the armrest, danger of entrapment.
The patient must not rock in the chair, because it may overturn.
Transfer the patient at a suitable speed.

10/ TECHNICAL SERVICE

Tecnimoem 97, S.L. has a dealer network for the sale of all its products. In the event of any problem, the end user must contact the dealer from which the product was purchased. Tecnimoem trains its dealers and provides them with sufficient spare parts to maintain the products.

 *TECNIMOEM 97, S.L. has a policy of continually improving its products. For that reason, device specifications may be changed without notice.*

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© TÜV Rheinland